


Breathable Train® Active Knit
gentle on the skin,
excellent wearing comfort

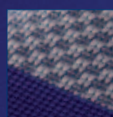

Three-dimensional Train® Active Knit
for a perfect fit



Less compression at edges prevents
constriction



Stretches easily
so support is easy to
put on and take off

Individually adjustable strap system
dynamic stabilization

**What's stopping you
from a life in motion?**

- You can use MalleoTrain® S preventatively to minimize the risk of injury.
- You don't have to give up sports just because you have a mild sprain or a torn ligament. MalleoTrain® gives your ankle reliable support and stability.

Performance and Prevention

Are your ligaments unstable?

- When they feel unstable, protect and support your ligaments with the MalleoTrain® S active support.

Stability and security when exercising

- The strap is wound around in a stabilizing figure-8 pattern, securing your ankle.
- When wrapping your ankle you can individually adjust the amount of stretch, determining the active support's effect.

Always ready for action...

MalleoTrain® S
can be put on in **15 seconds!**

Comfortable to wear ...

- The air-permeable, breathable knit is gentle on the skin and feels good to wear.
- The active support fits comfortably in all comfort shoes and athletic footwear.

MalleoTrain® S
The training partner
for your ankle.

SUPPORTS

BAUERFEIND®

BAUERFEIND®

The new MalleoTrain® S – goes on quickly and easily!



1

Put the active support on



2

Wrap the strap from the inside of
your foot under your foot.



3

Bring the strap diagonally
across the top of your foot.



4

Then wrap it around your
ankle above your heel ...



5

... and then again
diagonally toward the
front over the top of
your foot.



6

Wrap the thin part of the strap
under your foot to the outside
and then diagonally across the
top of your foot.



7

Stick the end of the
strap to the strap itself.

BAUERFEIND®

SUPPORTS AND ORTHOSES
MEDICAL COMPRESSION STOCKINGS
INSOLES AND SHOES
PROSTHETIC COMPONENTS AND MATERIALS
MEASUREMENT TECHNOLOGY

United Kingdom
Bauerfeind UK
Phyllis House
229 Bristol Road
GB-Birmingham B5 7UB
Phone +44 (0) 121 446 53 53
Fax +44 (0) 121 446 54 54
E-mail info@bauerfeind.co.uk

USA
Bauerfeind USA, Inc.
55 Chastain Road
USA-Kennesaw, GA 30144
Phone 1 800 423 34 05
Fax (770) 429 84 77
E-mail info@bauerfeindusa.com